

**LIFEPAK CR PLUS DEFIBRILLATOR
USER'S MONTHLY CHECKLIST**

Unit Serial No: _____

HSLP Inventory No: _____

Location: _____

Print the date and your initials in a column. Then place a checkmark as each item is completed.
Contact Heart Safe La Plata at 382-6039 if any problems are noted.
Fax the completed report to 884-7828.

Instruction	Recommended Corrective Action	Date												
		Initials												
1 Check readiness display for OK indicator CHARGE-PAK indicator ATTENTION indicator WRENCH indicator	None. Replace CHARGE-PAK and QUIK-PAK electrode packet. Refer to operating instructions. Contact authorized service personnel.													
2 Check Use By date on QUIK-PAK electrode packet	Replace electrode packet and CHARGE-PAK if date passed.													
3 Check additional supplies	Replenish as needed.													
4 Check defibrillator for Damage or cracks Foreign substances	Contact authorized service personnel. Clean the device.													
5 Other														

Does the world have you down? Feeling blue and lifeless? THIS is your answer! HSLP, we cure the blues!